


SecurA Cath™

PICC with Subcutaneous Securement System

Catheter Removal Procedure


- Remove dressing
- Grasp catheter hub near insertion site

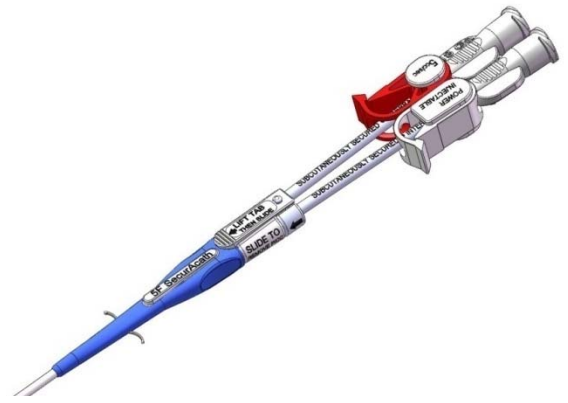

- Lift the flip-lock tab



- Move the flip-lock forward until it stops to retract the anchor

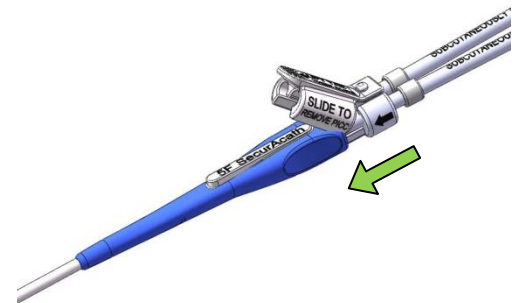

WARNING: Do not attempt to remove the catheter when the anchor is deployed

- Remove the catheter slowly. Do not use excessive force.
- If resistance is felt, stop removal. Apply warm compress and wait 20-30 minutes.
- Resume removal procedure. If further difficulty is experienced, obtain an x-ray and consult physician.

Please refer to instructions for use for indications, contraindications, hazards, warnings cautions and directions for use



181 Cheshire Lane Suite 100
Plymouth, MN 55441
763-225-6699
www.interradmedical.com